



Please refer to RVII Instruction Manual for details.

Examination Basic Flow

1. Start up and system initialization
2. Data transfer
 - 2.1 AUTOREF Data
 - 2.2 LM Data
3. PD and forehead rest adjustment
4. Exam
 - 4.1 Preset program
 - 4.2 Individual Exam.
5. Data comparison
6. Print and/or data to PC

Basic Flow


Setting Menu

Main Menu


- Simple Setting
- Eye Exam Program Edit
- Factory Default
- Initial Setting
- Back to previous

1. Start up and System Initialization


To initialize RVII system

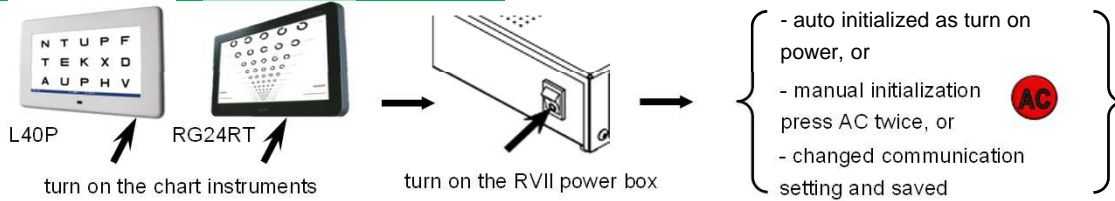


Plano Exam - VA Check

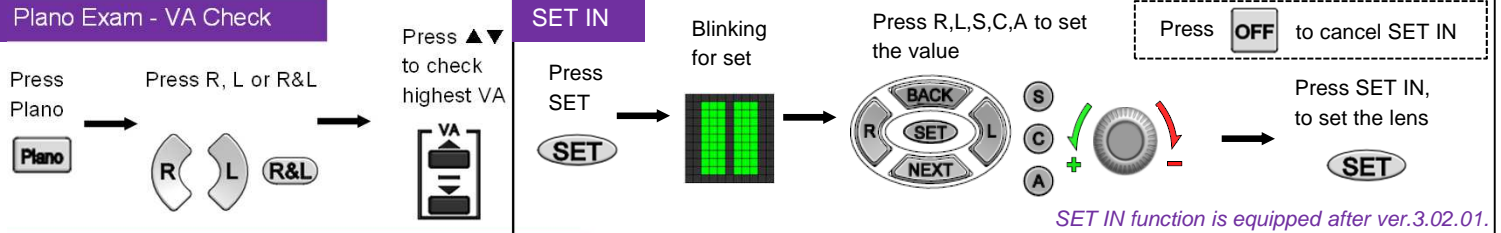


2.1 Data Input/Transfer from Autoref

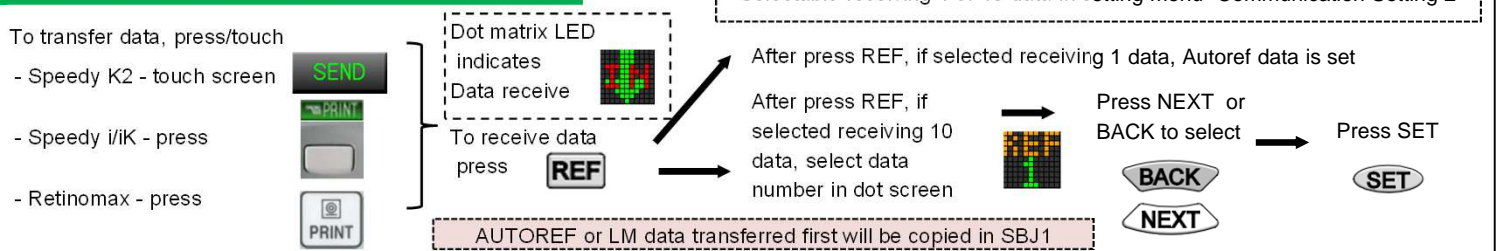


2.2 Data Input/Transfer from Lens Meter

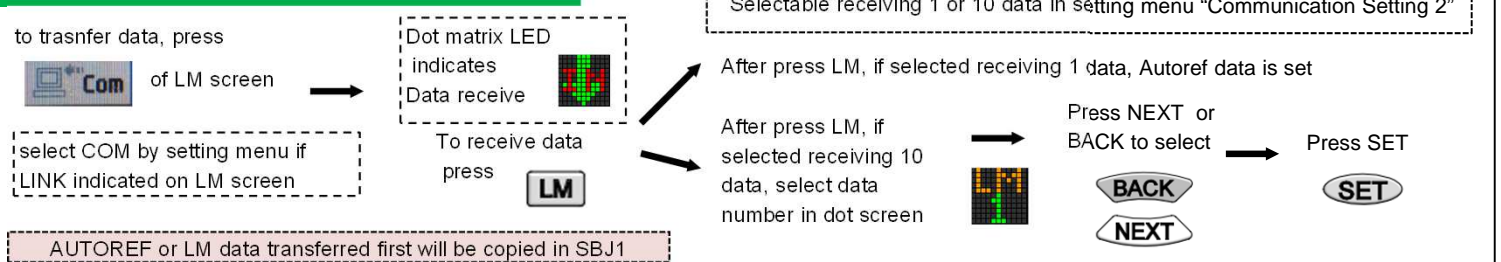


3. PD and Forehead Rest Adjustment

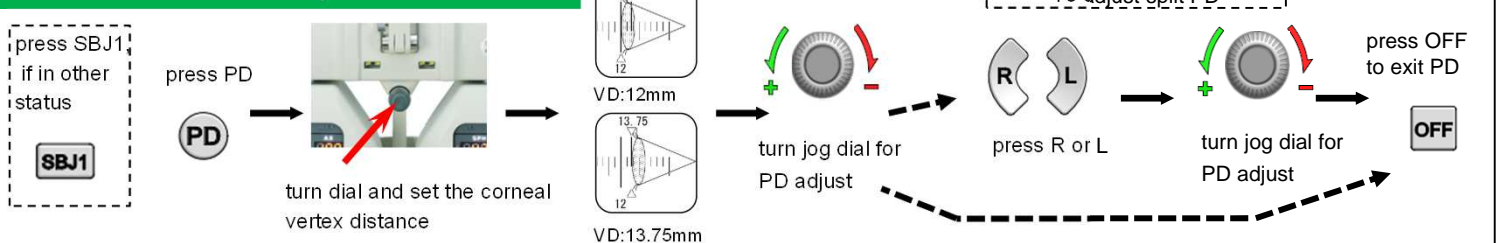


4.1 Preset Program EXAM.

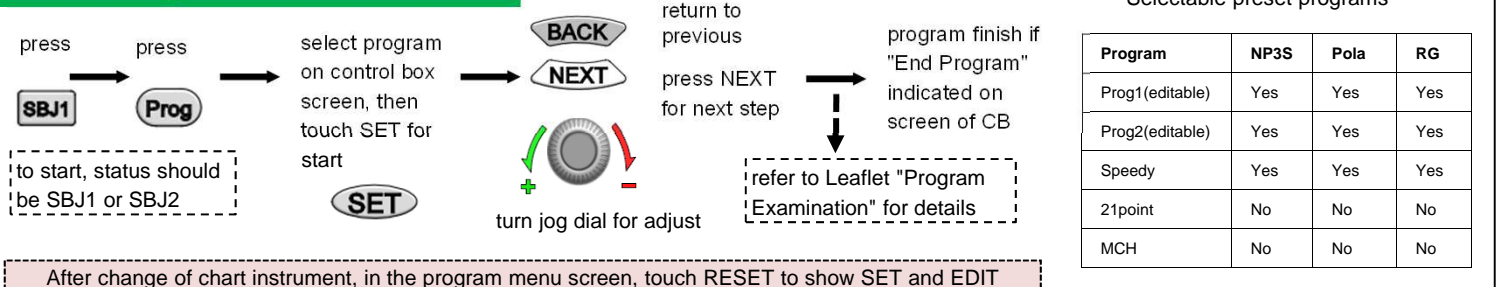


4.2.1 Astigmatism EXAM.

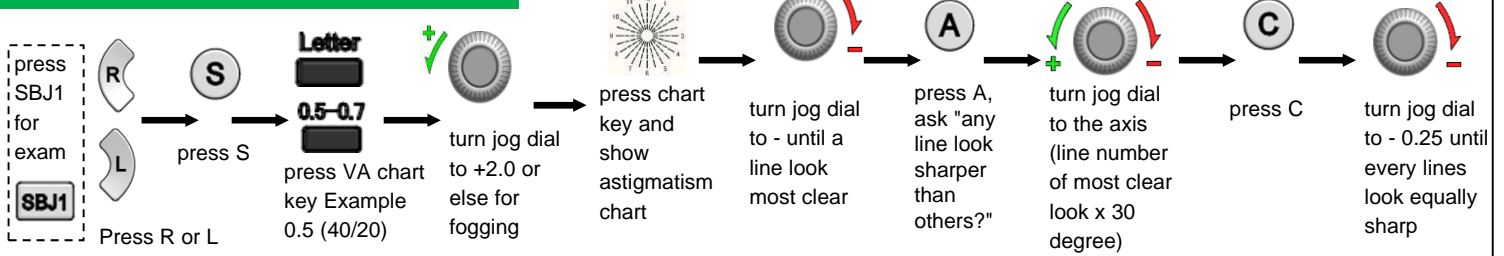

4.2.2 Red Green Test

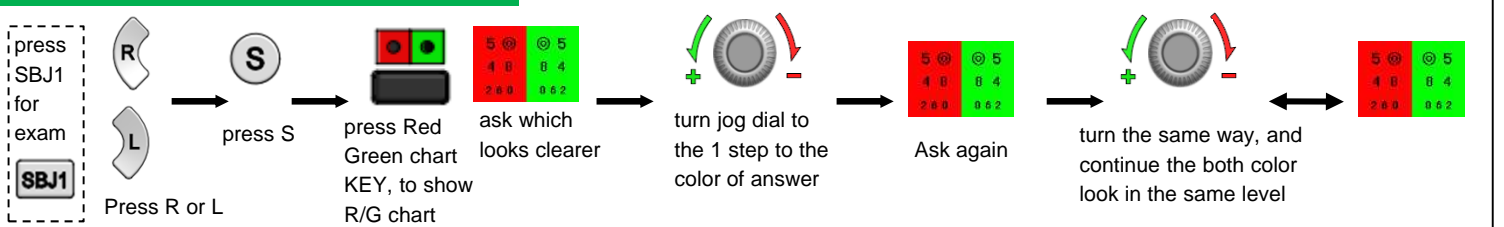


4.2.3 Cross Cylinder – Auto / Manual

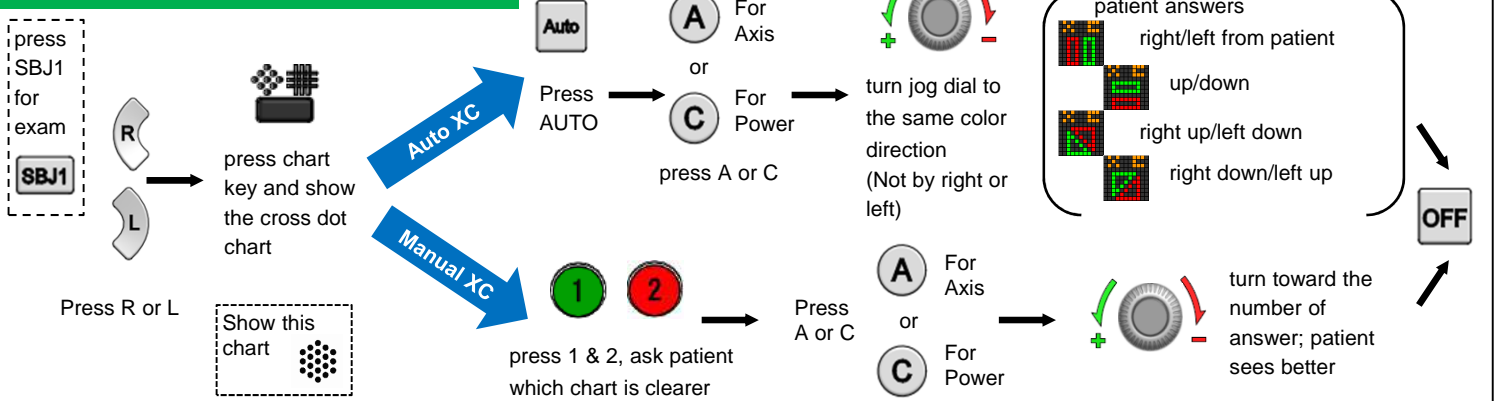


4.2.4 Extra Cross Cylinder (EXC)

Extra Cross Cylinder (EXC) is for: when the result of Cyl was 0 or use ref data, but considered need to check residual astigmatism, proceed EXC.

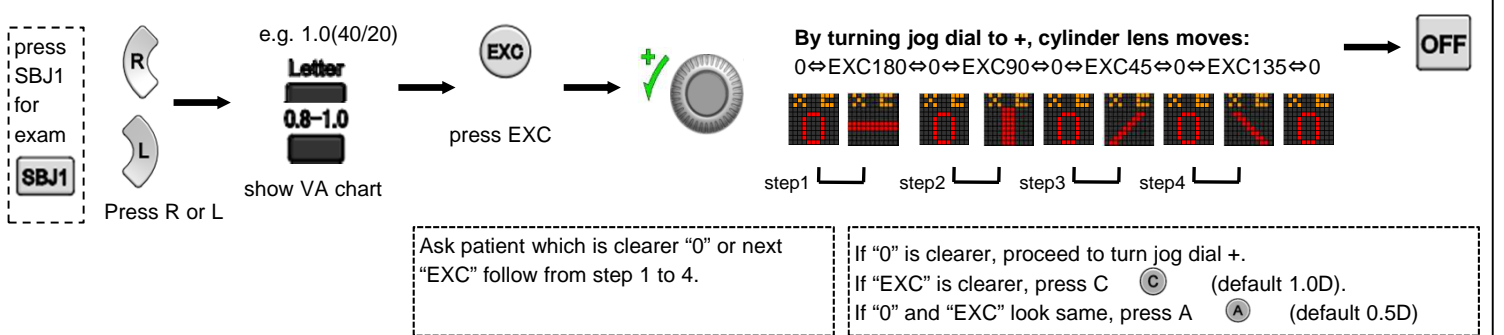

4.2.5 Near Point Test

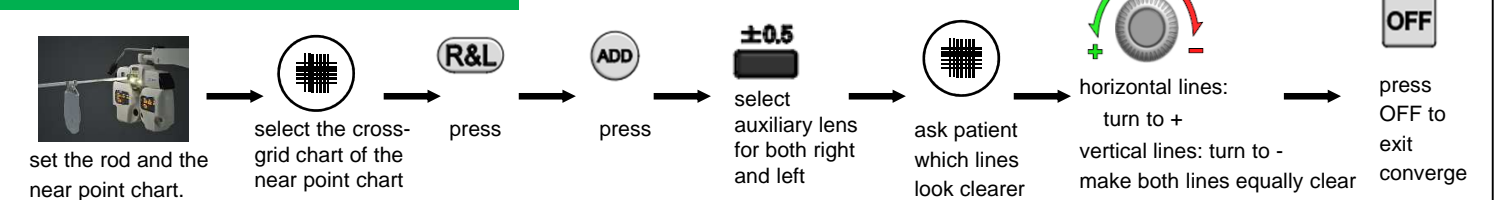

5. Data Comparison

press one of the button, which make blinking, to show the data

measured data in SBJ1 copy to SBJ2 for adjustment



6. Printout/Data to PC/Data Clear

PRT

Press for print all the data. Data remains after printing.

Available by setting: PRINT/PC/ PRINT+PC

Clear

Clear the indicated data only (Remote control)