

4.6 Cleaning

If the cover or the LCD monitor becomes contaminated, wipe them with a soft cloth. For severe stains, soak the cloth in a neutral detergent, wring well, and then wipe. Finally dry with a soft, dry cloth.

CAUTION

- Never use an organic solvent such as paint thinner or alcohol.
- Wipe the LCD touch-screen gently with a soft cloth. Never press hard objects against or place magnetic objects near the LCD touch-screen.
The surface of the LCD may be damaged. Device malfunction may also result.
- Never use a sponge or cloth soaked in water.
Water may leak into the interior of the device resulting in malfunction.

4.6.1 Cleaning the objective lens

Fingerprints or dust on the objective lens reduces the quality of the fundus images. Check the cleanliness of the objective lens before use. If the objective lens is contaminated, clean it.

- 1** Use the blower to blow off any dust from the objective lens.
- 2** Wrap the cleaning paper around a thin stick (or use a cotton swab) and dip the stick into the alcohol solution so that an appropriate amount soaks into the tip of the stick, then wipe the objective lens.

Note

- Do not use a hard stick such as one made of metal. Use something that will not scratch the glass.
- Starting from the center, gently wipe the lens by moving the stick outward in a spiral.

- 3** If the lens is still contaminated, use a new swab and repeat the above process until the lens is clean.